ESTETIC ESTETIC MAGAZINE

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INTIMATE
CHALLENGES AND
SUPREME POWER

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Beard and
moustache
transplants:
a new standard

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A SKIN LIFT TO REJUVENATE WITHOUT ALTERING VOLUMES

Aesthetic medicine treatments designed for men are fundamentally different to those designed for women.

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■ Skin type, bone structure, virility, the way the facial expressions are made and the patients' expectations all direct us towards a progressive treatment plan, which requires a very specific anatomical and psychological approach.

The demands of masculinity

In men, it is important not to alter the volumes, apart from if they have experienced significant weight loss or health issues, or if the patient is advanced in age and is having their first treatment later on in life. Men seek to improve their skin quality but do not want people to be able to see that they have undergone a medical procedure. Since male skin is thicker and therefore more affected by gravity than thinner skin, it needs more collagen and elastin to firm it up.

Firming in depth and at the surface

For this indication, I offer my patients a combined protocol. We start with an Ultherapy session, which allows us to set the foundations of the treatment. The micro-focused ultrasounds →

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are sent to three depths, which allows us to work on the subcutaneous muscle foundations with a first passage that penetrates to a depth of 4.5mm. The Deepsee® control screen allows us to see the targeted areas – such as the SMAS – in real time, which helps us carry out an accurate treatment. I pay extra attention to the mandibular angles, the chin and the submental area, which are what define the virility and dynamics of a masculine face. The second passage goes 3mm deep, into the mid and deep dermis, to activate fibroblast stimulation, which leads to the production of collagen and elastin. Finally, the last passage is at a depth of 1.5mm to work on the surface and boost the skin tightening and firming effect, as well as evening the complexion and boosting radiance. Like with any ultrasound treatment, the results appear gradually, and the final results are visible after 4 to 6

MALE SKIN IS THICKER AND THEREFORE MORE AFFECTED BY GRAVITY THAN THINNER SKIN, SO IT NEEDS MORE COLLAGEN AND ELASTIN TO FIRM IT UP.

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months. Only one session is required, which many men appreciate, as well as the fact that it does not affect the volumes and that the results are visible gradually, which fulfils their desire for discretion.

Skin lifting and enhancing virility

This action will be reinforced a month later with an injection of Radiesse®, which I believe to be the best tissue inducer, not only because of its collagen and elastin bio stimulation, but also because it has proven efficacy on angiogenesis – backed up by published clinical studies – which is essential for vascularisation and skin trophicity.

For this indication, I dilute it 1:1 or 1:2 with saline solution, depending on the skin's thickness. For men with very thick or damaged skin, I schedule a second session after three months in order to enhance the results and make them last longer. The injections are performed using a cannula with an entry point in the cheekbone and one in the angle of the jaw. The aim is always to respect and highlight the angular nature of the male face. These retracing injections are performed in a fan shape to distribute the product evenly and create lines of tightened tissue, which offer a subtle lifting effect. For indications that involve making the lower part of the face look more masculine, we can inject undiluted Radiesse® using a cannula in a linear retracing pattern into the angle of the jaw and the jawline, as well as bringing forward and reshaping the jaw by placing boluses of product with a needle.

More varied patient profiles

I am treating ever younger patients, who are aware of the early signs of sagging skin. They want to improve their skin's tonicity and their facial dynamics, and they are aware that the preventative action of these treatments can make them look younger for longer. These men – who often come from highly competitive backgrounds such as banking, start-ups or high-up in industry or communications – particularly appreciate the option of having a straightforward treatment with no side effects, carried out in 2 or 3 sessions, with long-lasting effects that can be accumulated over the medium and long term. These treatments are very popular due to the gradual appearance of their effects, as the patient's entourage cannot tell that they have undergone a medical procedure. This perfectly fulfils their demand for efficacy, discretion and durability. ■

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Aesthetic and anti-aging medicine. Laureate and medallist from Paris medical faculty, former Registrar at Paris Descartes university. Diploma from the International College of Aesthetic Medicine. Member of the French Society of Morphological and Anti-aging Medicine, and the Francophone Society of Medical Lasers.

drbenjamincozanet.com [drbenjamincozanet](https://www.facebook.com/drbenjamincozanet) [drbenjamincozanet](https://www.instagram.com/drbenjamincozanet)